



NOVUS™
Nasal CPAP mask



Homecare • Novus nasal CPAP mask




Novus™ nasal CPAP mask

The Novus nasal CPAP mask is a comfortable and lightweight interface intended for use by adult patients (>30 kg) who have been prescribed CPAP or bilevel therapy in the home.

Designed to provide a well tolerated and easy solution for home care patients, Novus has a small number of parts, which is beneficial when fitting and adjusting the mask, and ensures that assembly and disassembly for cleaning is simple in the home environment.

Novus is available in two sizes, with a soft silicone seal, offering a comfortable and effective fit, suitable for the majority of users.

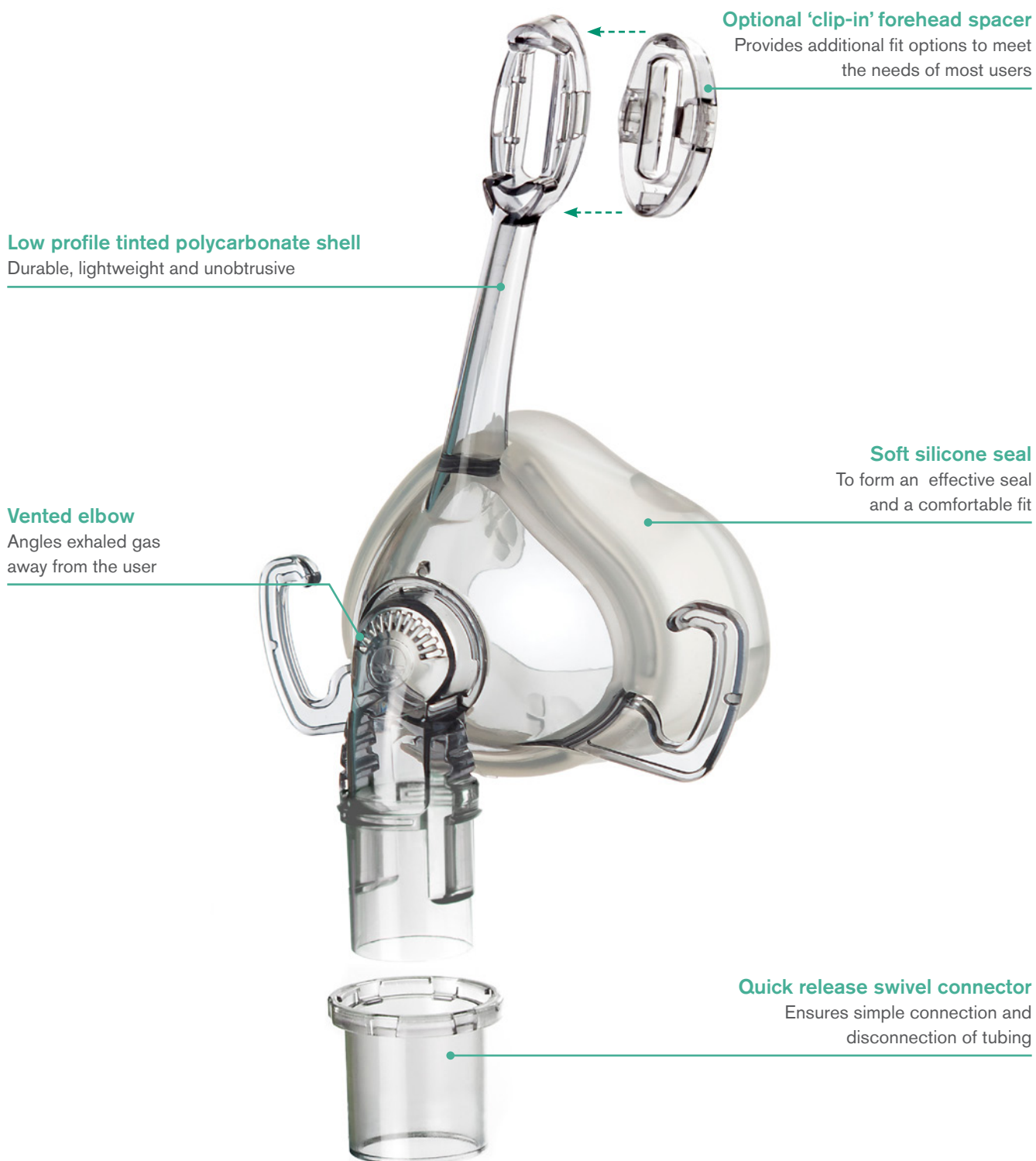




Novus™ nasal CPAP mask

Breathable headgear
Comfortable for the user

Optional quick release grip

Improves headgear fitting and can be added to either side of the mask, to suit all users

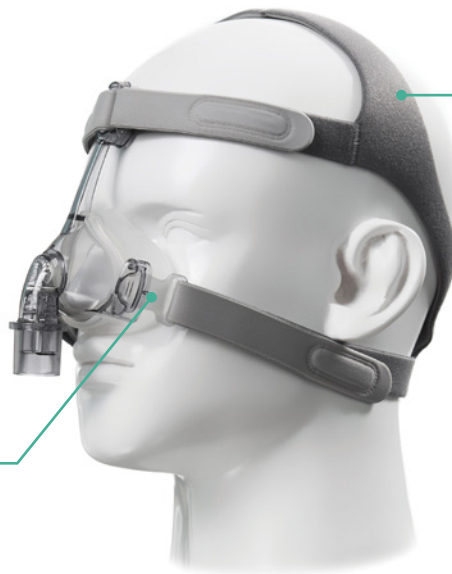

Ordering information

The Novus nasal CPAP mask is available in two sizes medium and large.

Code	Description	Languages in the Instructions For Use	Box Qty.
7772000	Novus nasal CPAP mask, Large	Chinese Traditional, Chinese Simplified, Czech, Japanese, Lithuanian, Russian, Turkish	8
7792000	Novus nasal CPAP mask, Medium	Chinese Traditional, Chinese Simplified, Czech, Japanese, Lithuanian, Russian, Turkish	8

Accessories

Code	Description	Box Qty.
7776000	Novus nasal CPAP mask headgear (including headgear and quick release headgear clip)	20
5318000	22 mm Smoothbore polypropylene tube with soft end connectors, 22F-22F, 1.8 m	50
5418000	15 mm Smoothbore polypropylene tube with soft end connectors, 22F-22F, 1.8 m	60



Mask fitting

How to obtain a leak free and comfortable mask fit.



1. Remove the swivel connector from the mask by pressing the connector's quick release tabs.



2. Insert the swivel connector into your device tubing.



3. Support the headgear and mask.



4. Place the mask seal over the nose and hold in position. Gently pull the headgear over the head ensuring that the lower straps are below the ears.



5. Check that the straps are not twisted and engage the remaining lower strap onto the mask hook.



6. Adjust the mask fitting by unfastening the upper straps, pulling both evenly and reattaching to the headgear. The fit should be secure but not over tight.



7. Complete the mask fitting by unfastening the lower straps, pulling both evenly and then reattaching. The fit should be secure but not over tight.



8. Ensure that the CPAP/bilevel device is operating. Air should be flowing from the device tubing. Attach the assembled swivel connector and device tubing to the mask elbow by clicking into place.



9. If required readjust the headgear straps to eliminate any air leaks from the mask seal. **Do not over tighten the mask.** If leaks persist lift the mask seal from the nose and reseat before readjusting the headgear straps.

IS1.44_R3 • Issue 3 11.24



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK
T: +44 (0)118 965 6300 info@intersurgical.com www.intersurgical.com



The manufacturer Intersurgical Ltd is certified to ISO 14001:2015, ISO 9001:2015, ISO 13485:2016 and MDSAP

Please think before you print
Save energy and paper.
If you must print this information sheet please print it double sided.