



STARMed hood kit for NIV and CPAP use


Quick set up guide



Critical Care ▪ Patient Interfaces ▪ STARMed



StarMed CaStar hood kit for NIV and CPAP use

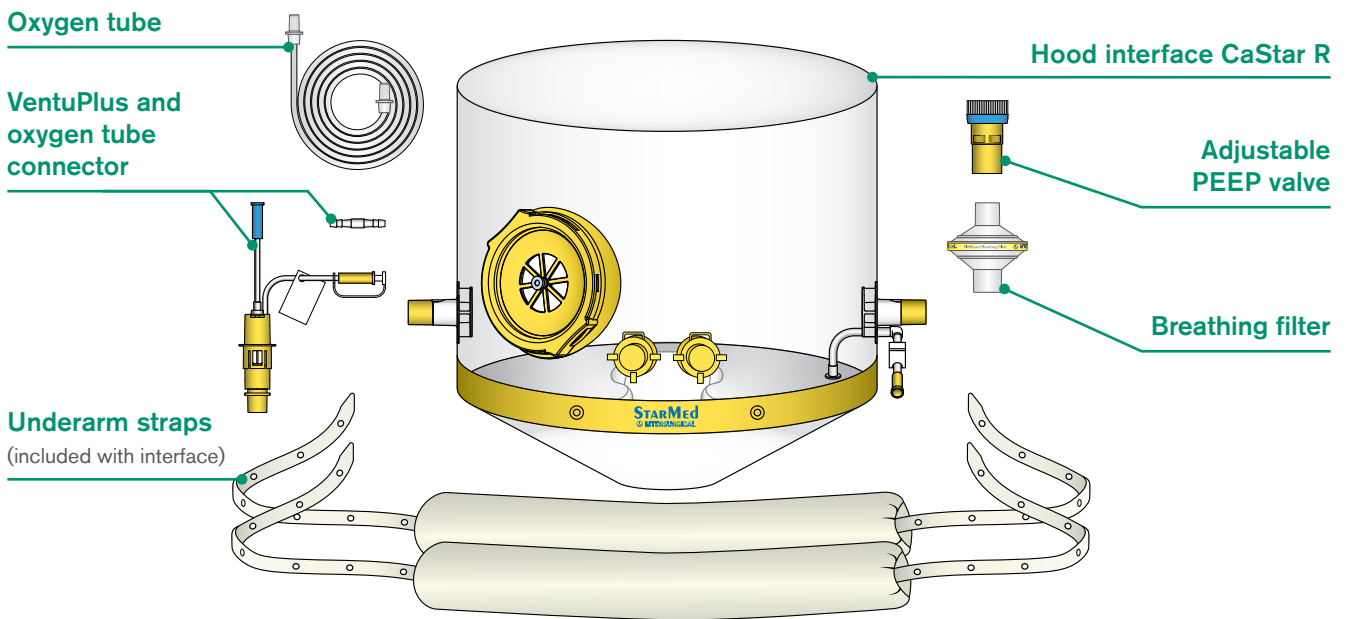

The CaStar R hood is a comfortable, versatile and lightweight patient interface for NIV. It is designed to offer an efficient and comfortable alternative to traditional methods of non-invasive respiratory support, it is well tolerated for extended periods and can reduce the risks associated with both endotracheal intubation and face mask use.

Generally used on a mechanical ventilator for delivering NimV (non-invasive mechanical ventilation), the CaStar R is also a versatile interface that can be used to provide optimal CPAP therapy via a constant flow generator and a mechanical PEEP valve.

We have provided a complete kit to allow for the flexible delivery of the required therapy: the same hood can be used on the mechanical ventilator for NimV, or it can be used for CPAP via external equipment or with the convenient Venturi valve accessory.

The kit includes:

- CaStar R NIV hood with bi-directional anti-asphyxiation valve and inflatable neck cushion
- VentuPlus Venturi flow driver accessory and an O₂ connector and tube
- Breathing filter
- Adjustable PEEP valve 0-20 cmH₂O with 22F connection

CaStar R hood kit for NIV and CPAP use with Venturi flow generator, breathing filter and PEEP valve


Code	Size	Neck circumference	Box Qty.
CP1170XXL/2R	Extra Extra Large	≥50cm	5
CP1170XL/2R	Extra Large	45-52cm	5
CP1170L/2R	Large	40-47cm	5
CP1170M/2R	Medium	34-41cm	5
CP1170S/2R	Small	27-34cm	5
CP1170XS/2R	Extra Small	17-27cm	5

The CaStar R hood kit includes the following Intersurgical products. These can also be ordered separately. Please refer to our website for the complete range of accessories.


Code	Description	Box Qty.
CP211*/2R	CaStar R hood with inflatable neck cushion, bi-directional anti-asphyxiation valve and apatient access port (* to be replaced with size letter XXL, XL, L, M, S, XS)	5
CP924	VentuPlus Venturi flow driver accessory	10
1174000	Oxygen tube, 1.8m	125
1690000	Flo-Guard breathing filter	50
SPV22F	Adjustable PEEP valve 0-20 cmH ₂ O with 22F connection	10




CPAP set up one: Use with dual O₂ flow meter

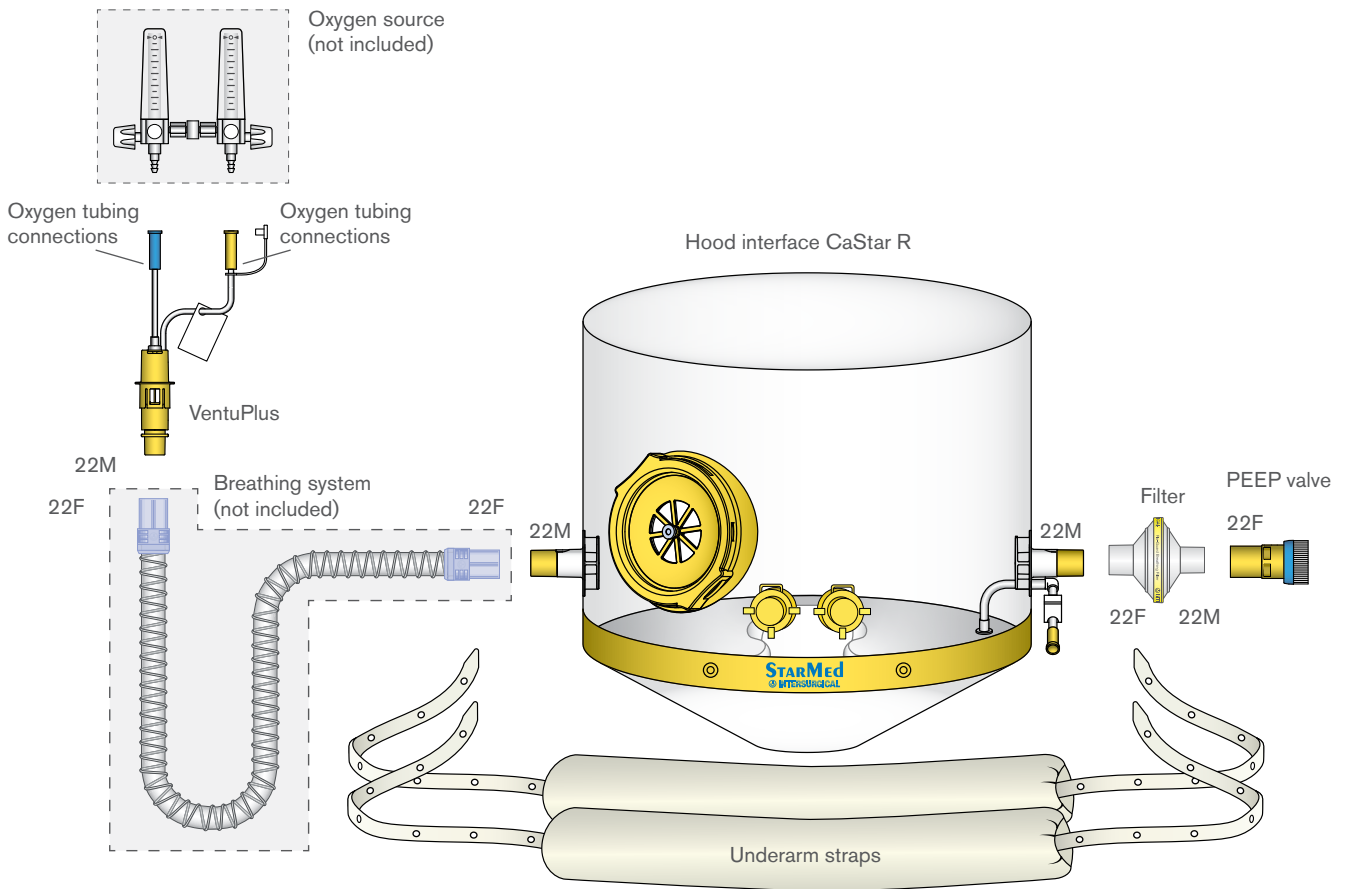


CPAP set up two: Use with two single O₂ flow meters

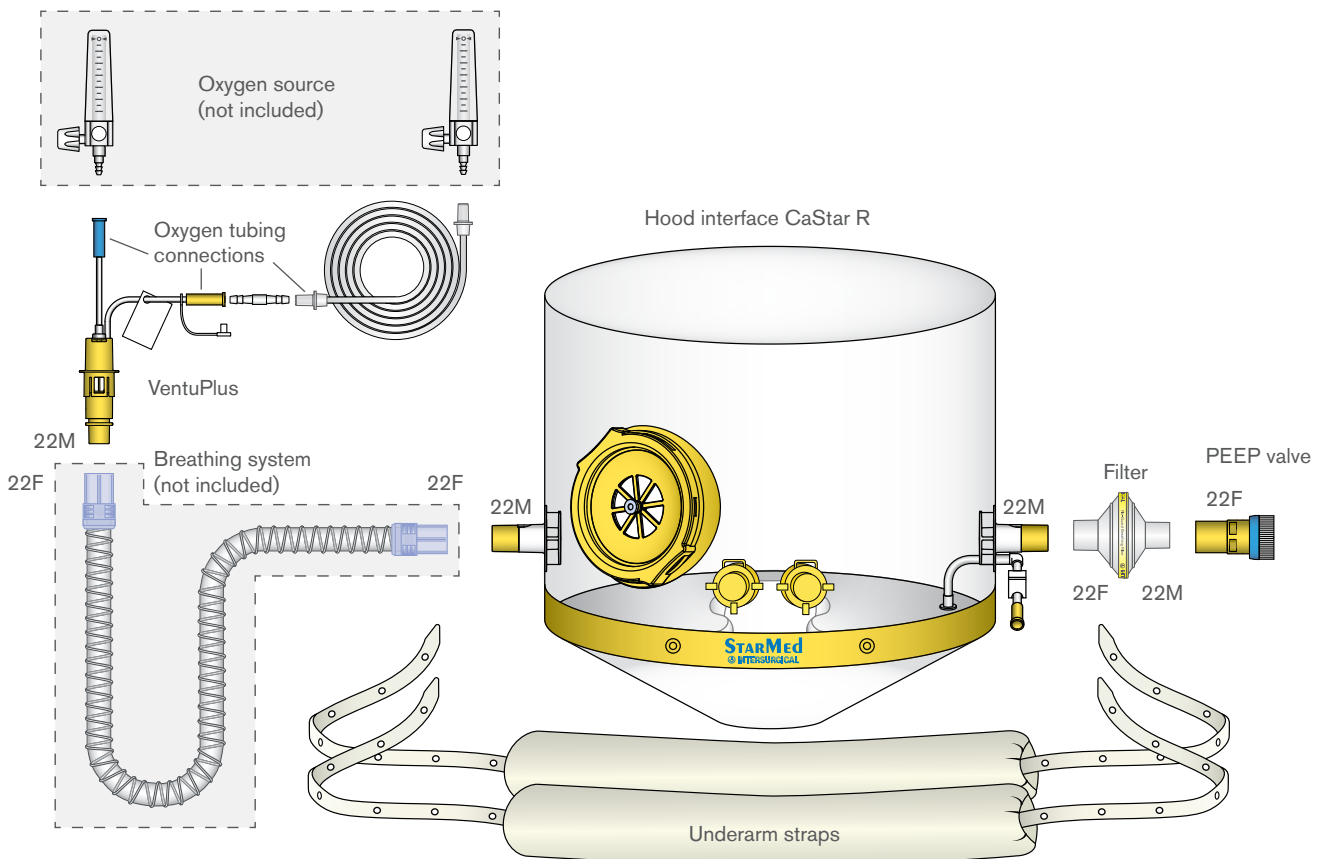

CPAP set up three: Use with single O₂ flow meter (delivered O₂% = 40%)

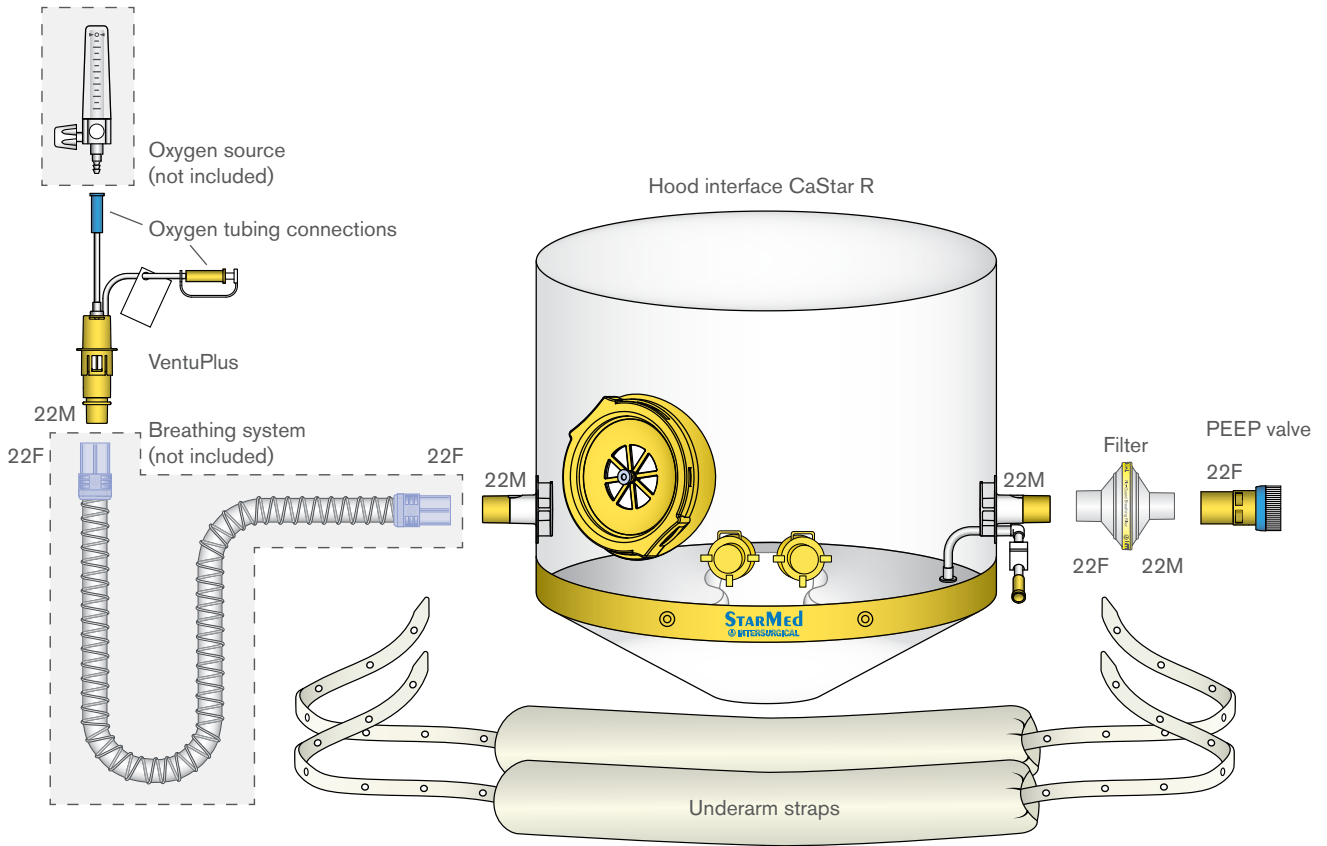


CPAP set up four: Use on CPAP flow driver

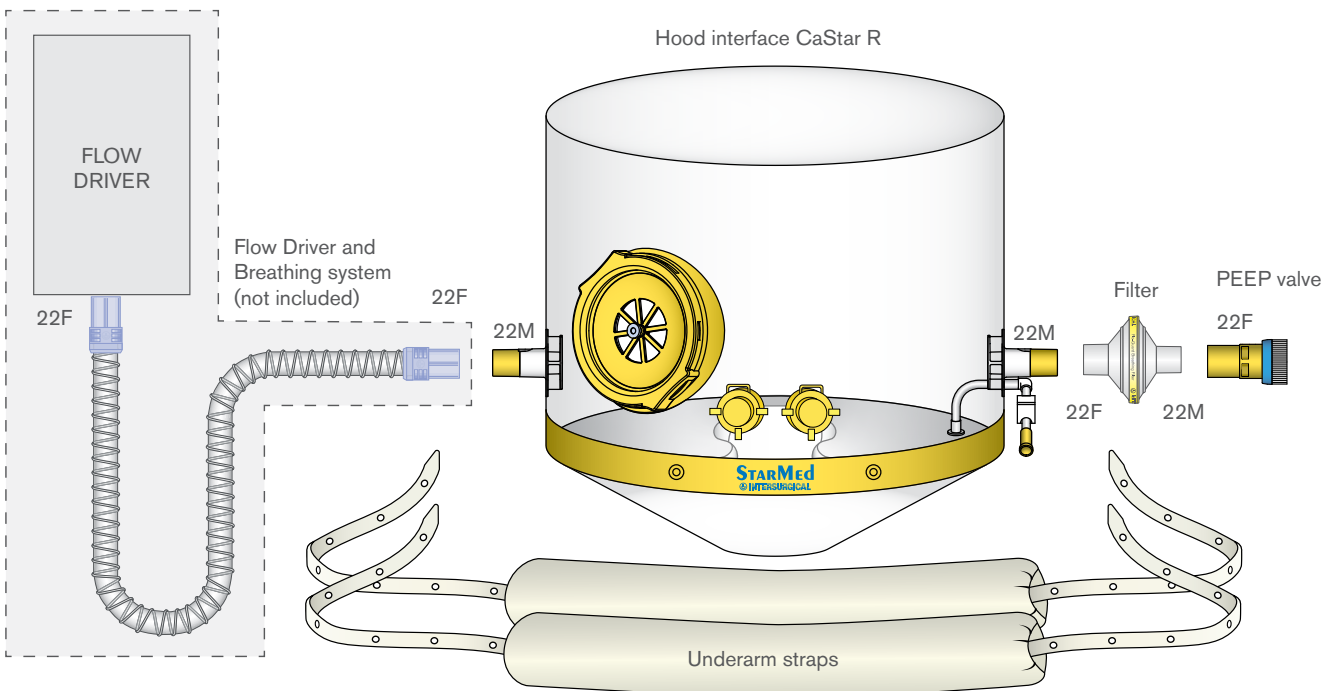



NIV set up: Use on dual limb ICU mechanical ventilator

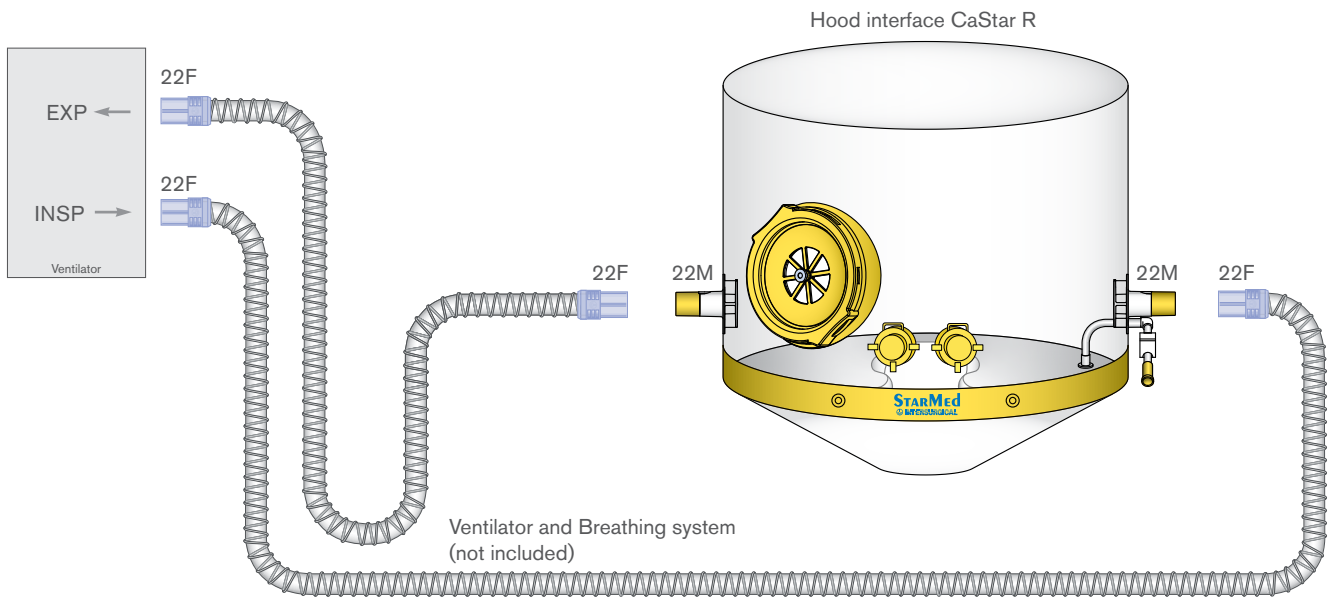

▶ CaStar R hood video available at www.intersurgical.com

Make an enquiry

StarMed landing page:

 www.intersurgical.com/info/starmed

StarMed evidence database:

STARMed
Respiratory hoods
and masks
evidence database



starmedevidence.intersurgical.com

User Guide video:

Watch the CaStar R Hood kit with VentuPlus video





Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK
T: +44 (0)118 965 6300 F: +44 (0)118 965 6356 info@intersurgical.com www.intersurgical.com



Management System
ISO 9001:2015
EN ISO 13485:2016



www.tuv.com
ID 900005840

Manufacturer:
Intersurgical S.p.A.
Mirandola (MO) Italy

Please think before you print
Save energy and paper. If you must print this
information sheet please print it double sided.

IS1.55_INT ▪ Issue 1 03.21