



GO-PAP™

Disposable CPAP device



Critical Care ▪ Pulmodyne®

GO-PAP™

The GO-PAP is a disposable emergency CPAP device that connects directly to an oxygen source. It operates at 10 L/min and delivers approximately 30% FiO₂. There is an adjustable PEEP valve (5, 7.5 and 10 cm H₂O) integrated into the generator. The PEEP is independent of the flow and therefore

there is no requirement to adjust the flow to maintain PEEP levels. Additionally, this completely disposable system offers integrated nebulisation while maintaining a consistent flow to the patient.

Two different headgear designs

standard or CAP headgear (as shown)

40 minutes of run time
with D-Cylinder tank

Adjustable PEEP
5/7.5/10 cm H₂O

Delivers approximately 90 L/min patient flow
when supplied with 10 L/min oxygen

Integrated nebulisation



GO-PAP System with nebuliser and BiTrac™ ED mask with CAP headgear

Code	Description	Box Qty.
313-4605N	GO-PAP System with oxygen tubing, nebuliser and BiTrac ED mask with CAP headgear, large adult	10
313-4604N	GO-PAP System with oxygen tubing, nebuliser and BiTrac ED mask with CAP headgear, medium adult	10

Make an enquiry



GO-PAP™ System and BiTrac™ ED mask with CAP headgear

Code	Description	Box Qty.
313-4605	GO-PAP System with oxygen tubing and BiTrac ED mask with CAP headgear, large adult	10
313-4604	GO-PAP System with oxygen tubing and BiTrac ED mask with CAP headgear, medium adult	10

Make an enquiry



GO-PAP System with nebuliser and BiTrac ED mask with standard headgear

Code	Description	Box Qty.
313-4603N	GO-PAP System with oxygen tubing, nebuliser and BiTrac ED mask, large adult	10
313-4602N	GO-PAP System with oxygen tubing, nebuliser and BiTrac ED mask, medium adult	10
313-4601N	GO-PAP System with oxygen tubing, nebuliser and BiTrac ED mask, small adult	10

Make an enquiry



GO-PAP System and BiTrac ED mask with standard headgear

Code	Description	Box Qty.
313-4603	GO-PAP System with oxygen tubing and BiTrac ED mask, large adult	10
313-4602	GO-PAP System with oxygen tubing and BiTrac ED mask, medium adult	10
313-4601	GO-PAP System with oxygen tubing and BiTrac ED mask, small adult	10

Make an enquiry

GO-PAP™ accessories

The BiTrac™ ED mask can be supplied separately to integrate the patient interface size range for GO-PAP. This mask features a dual lip seal and the adjustable Omni-Clip™ to guarantee optimal fit. The connection with the breathing system is a 30M/22F port that fits directly to the GO-PAP patient connections as well as to the ADAPT-2-VENT™ elbows accessories.



BiTrac ED mask with CAP headgear

Code	Description	Box Qty.
313-7030C	BiTrac ED mask with CAP headgear, large adult	10
313-7029C	BiTrac ED mask with CAP headgear, medium adult	10
313-7028C	BiTrac ED mask with CAP headgear, small adult	10

Make an enquiry



BiTrac ED mask with standard headgear

Code	Description	Box Qty.
313-7031	BiTrac ED mask, extra large adult	10
313-7030	BiTrac ED mask, large adult	10
313-7029	BiTrac ED mask, medium adult	10
313-7028	BiTrac ED mask, small adult	10

Make an enquiry

ADAPT-2-VENT and more accessories

Accessories include the ADAPT-2-VENT which is an adapter elbow that allows the use of the BiTrac ED mask on standard dual limb or single limb ventilator circuits. This allows the patient to move from CPAP, using the GO-PAP, to other forms of ventilation without changing the patient interface. The 18M nebuliser that fits the GO-PAP can be supplied separately as well as the GO-PAP generator.



Code	Description	Box Qty.
313-4600	GO-PAP System generator with oxygen tubing	10
313-9185	ADAPT-2-VENT 30 mm F non-vented elbow with anti-asphyxiation valve	10
313-9186	ADAPT-2-VENT 30 mm F standard non-vented elbow	10
1402018	Cirrus™2 nebuliser 18M with tube, 1.8 m	45

ADAPT-2-VENT, BiTrac, GO-PAP and Omni-Clip are manufactured by Pulmonary Inc and distributed by Intersurgical Ltd.

Note: Not all products are available in all markets, please contact your local office for details.

IS1.63 ▪ Issue 5 02.26



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK
T: +44 (0)118 965 6300 info@intersurgical.com www.intersurgical.com